CHROMA ZONE™ COLOR STAIN

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I. INTRODUCTION

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Catalysts
 Accelerators

VI. CLINICAL PROCEDURES
VI-1. Curing unit and polymerization time
 The recommended effective wavelength of dental curing unit is 400- 515 nm.
 Table: Light curing time

Dental curing unit	Light curing time
Dentacolor XS	30 seconds

Curing characteristics

Color	*Depth of cure (mm)
	Dentacolor-XS
Clear	4.52
White	1.25
Dark Brown	0.28
Clear Paste	1.73

* The depth of cure was measured by Kuraray Noritake Dental Inc. according to ISO 4049.

VI-2. Basic laboratory procedures
1. Staining facing crown for crown and bridge materials
 1) Fabrication of a metal frame
 Fabricate a metal frame in the usual manner.
 2) Application and light curing Opaque resin
 Apply Body Opaque paste (e.g. EPRICORD, ESTENIA C&B) to the facing surface of metal frame. If necessary, apply Opaque Modifier to the cervical area to adjust the color and light cure it according to the manufacturer's instruction.
 3) Forming and preliminarily light curing Cervical and Dentin
 Form Cervical and Dentin paste, and light cure the paste preliminarily according to the manufacturer's instruction.
 4) Shade selection of CHROMA ZONE COLOR STAIN
 Select an appropriate shade of STAIN to adjust the color.
 5) Staining Cervical and Dentin
 Apply STAIN and light cure it with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".
 6) Forming the Incisal edge
 Form the incisal edge using a mixture of STAIN and CLEAR PASTE to reproduce the transparency required. And light cure it with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time". Apply Amber paste thinly on the incisal edge and Transparent paste in the proximal area, then perform the final polymerization.
 7) Completion
 Shape and polish in the usual manner to complete the crown.

2. Staining inlays and onlays
 1) Fabrication of a working model
 Fabricate a working model in the usual manner.
 2) Application and drying of a resin separating agent
 Apply a resin separating agent inside the cavity on the working model, and dry it.
 3) Forming and light curing Dentin
 Form Dentin paste (e.g. EPRICORD, ESTENIA C&B) on the cavity, and light cure the paste according to the manufacturer's instruction.
 4) Shade selection of CHROMA ZONE COLOR STAIN
 Select an appropriate shade of STAIN to adjust the color.
 5) Staining Dentin
 Apply STAIN to the occlusal area or the cervical area.
 6) Light curing the STAIN
 After the application, light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".
 7) Forming and curing Enamel
 Apply Enamel paste to the cured STAIN, and light cure it according to the manufacturer's instruction.
 8) Shaping and polishing
 Shape and polish in the usual manner to complete the inlay or the onlay.
 9) Staining pits and fissures
 Stain the pits and the fissures, and light cure STAIN with a dental curing unit (See table "Light curing time") for the specified length of time shown on the table "Light curing time".

WARRANTY
 Kuraray Noritake Dental Inc. will replace any product that is proved to be defective.
 Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]
 "CHROMA ZONE", "EPRICORD" and "ESTENIA" are trademarks of KURARAY CO., LTD.
 "Dentacolor" is a trademark of Heraeus Kulzer GmbH.

Kuraray Noritake Dental Inc.
 1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Kuraray Europe GmbH
 Philipp-Reis-Str. 4,
 65795 Hattersheim am Main, Germany
 Phone:+49 (0)69 305 35 840 Fax:+49 (0)69 305 35 640

Kuraray Noritake Dental Inc.
 1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Kuraray Europe GmbH
 Philipp-Reis-Str. 4,
 65795 Hattersheim am Main, Germany
 Phone:+49 (0)69 305 35 840 Fax:+49 (0)69 305 35 640

Kuraray Noritake Dental Inc.
 1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Kuraray Europe GmbH
 Philipp-Reis-Str. 4,
 65795 Hattersheim am Main, Germany
 Phone:+49 (0)69 305 35 840 Fax:+49 (0)69 305 35 640